


TABLE OF CONTENTS

Figure		Page
1	Light Models	2
2	Wall Model Light Assembly	4
3	Floor Model Light Assembly	6
4	Ceiling Model Light Assembly	8
5	Track Model Light Assembly.....	10
6	Lighthead Assembly	14
7	Reflector Assembly.....	18
8	Counterbalance Arm Assembly	20
9	Base Assembly (Floor Light)	22
10	Transformer Assembly (Floor Light - Export)	24
11	Single Point Mounting Suspension Package	26
12	Trolley Assembly & Track Assembly.....	28
13a	Light Control Assembly (120V-600W)	33
13b	Light Control Assembly (230V-300W)	34
	Recommended Spare Parts List	36

LIGHT MODELS

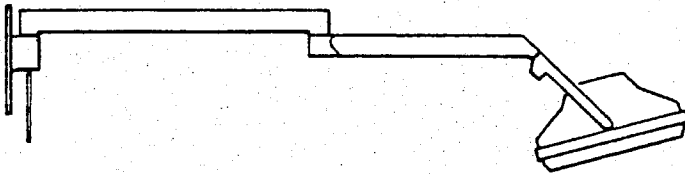


- 2310** Wall Model Examination Light (120V)
- 2311** Wall Model Examination Light (120V Export)
- 2410** Wall Model Surgical Light (120V)
- 2411** Wall Model Surgical Light (120V Export)
- 2412** Wall Model Surgical Light (230V Export)

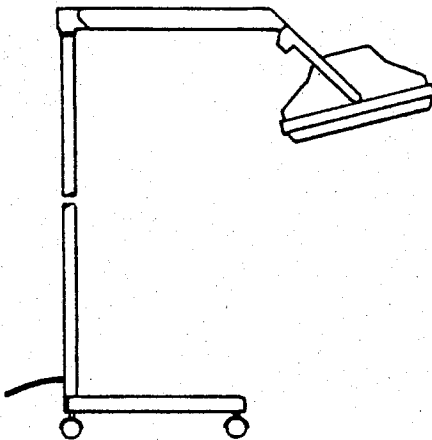


- 2310** Portable Examination Light (Floor - 120V)
- 2311** Portable Examination Light (Floor - 120V Export)
- 2312** Portable Examination Light (Floor - 230V Export)
- 2410** Portable Surgical Light (Floor - 120V)
- 2411** Portable Surgical Light (Floor - 120V Export)
- 2412** Portable Surgical Light (Floor - 230V Export)

LIGHT MODELS

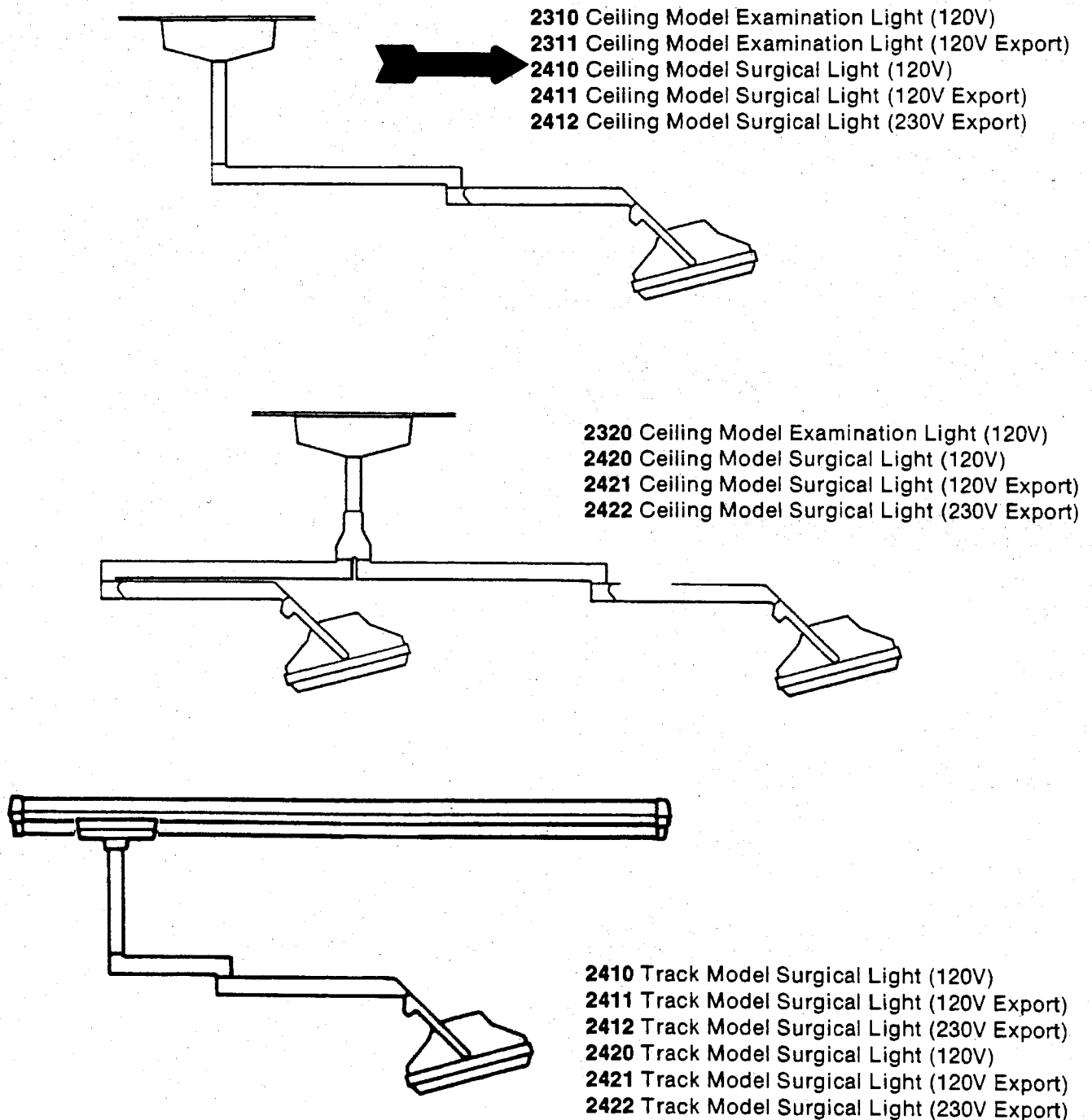

SURGICAL & EXAMINATION LIGHTS **Figure 2. Wall Model Light Assembly**

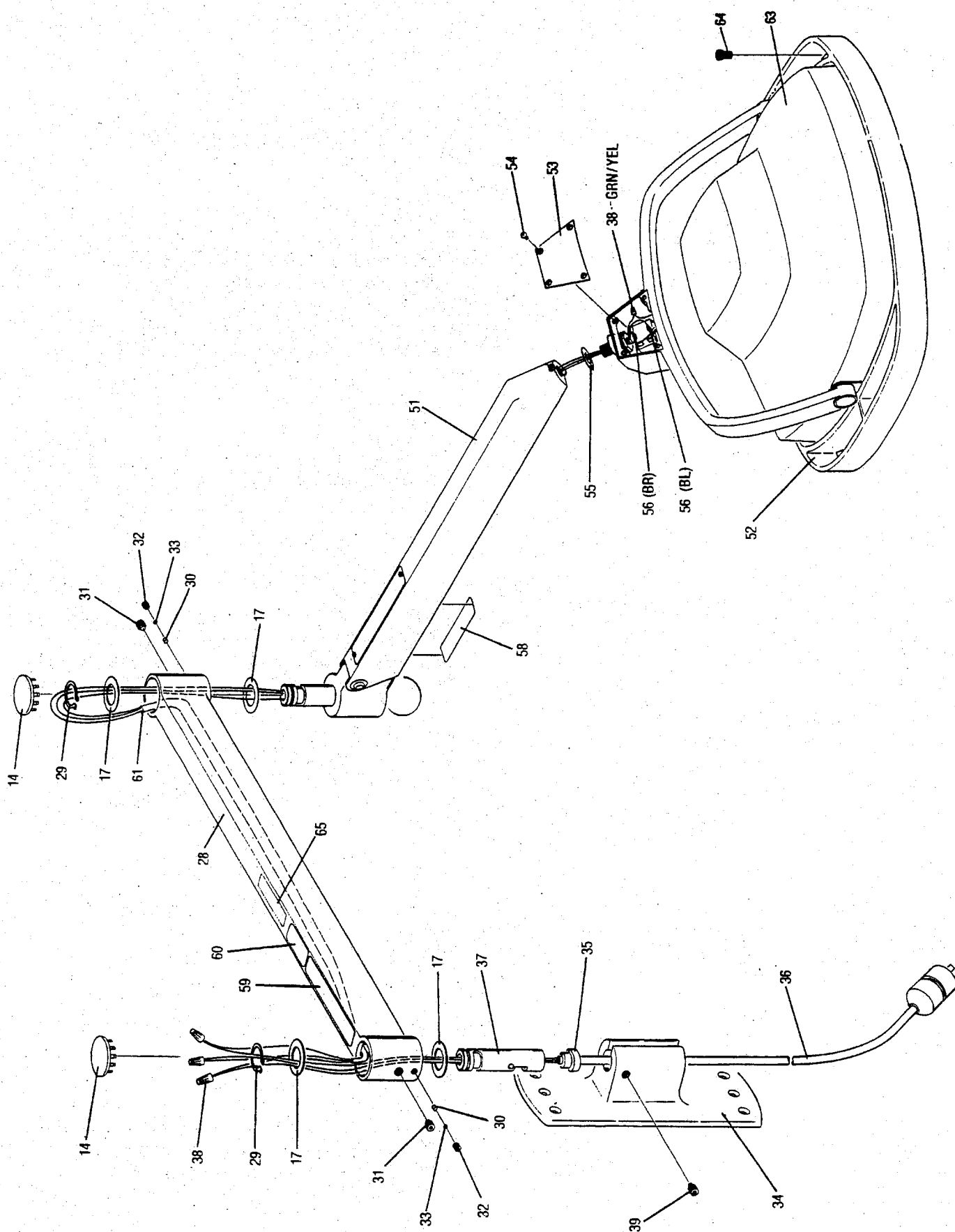


REF. 65129R

Wall Model Light Assembly — Refer to Figure 2

ITEM	PART NO.	DESCRIPTION
14	65059	Button, Plug for 1-7/16 dia Hole
17	65061	Race, Thrust, 7/8 ID
28	65157	Arm, Wall Light Extension (Brown)
	65158	Arm, Wall Light Extension (White)
29	E6207	Ring, External Retaining for 7/8 dia Shaft
30	43249	Plug, Friction, 3/16 dia x 1/8 long
31	E6244	Screw, 3/8-16 x 1/2 Half Dog Point Socket Set (with Nylon Insert)
32	E6981	Screw, 1/4-20 x 3/16 Flat Point Socket Set (with Nylon Insert)
33	65058	Washer, 15/16 Spring
34	65166	Bracket, Wall Mounting (Brown)
	65167	Bracket, Wall Mounting (White)
35	57030	Strain Relief, 21/64 - 23/64 dia Cable
36	65095	Cord, Power Supply
37	65023	Pin, Extension Arm Rotation
38	B0105	Nut, Wire (Two #18 Min - Two #16 Max)
39	E2212	Screw, 5/16-18 x 1/2 Dog Point Socket Set
51	65111	Arm Assembly, Counterbalance (Brown)
	65172	Arm Assembly, Counterbalance (White)
		• See Figure 8 for listing of parts
52	65085	Lighthouse Assembly, 100W (Examination)
	57651	Lighthouse Assembly, 150W (Surgical)
		• See Figure 6 for listing of parts
53	65175	Cover, Switch (Castle Surgical Light)
	511772	Cover, Switch (Castle Examination Light)
	—	Cover, Switch (Model 2300 Light)
54	E6876	Screw, #6-32 x 5/16 Self-Tapping Phillips Oval Head
55	65026	Washer, Keyed
56	65199	Terminal, Female Slip-On Flag
58	65141	Decal, Danger, Explosion Hazard
59	301110	Decal, Light Rating
60	5116100009	Label, ETL-IEC
61	R1922	Tubing, Plastic (specify length)
63	513915	Cover, Lighthouse
64	E6333	Screw, #6-32 x 3/8 Hex Socket Head
65	46018	Plate, Patent
—	65196	Handle, Sterilizable (Optional)

SURGICAL & EXAMINATION LIGHTS
Figure 3. Floor Model Light Assembly

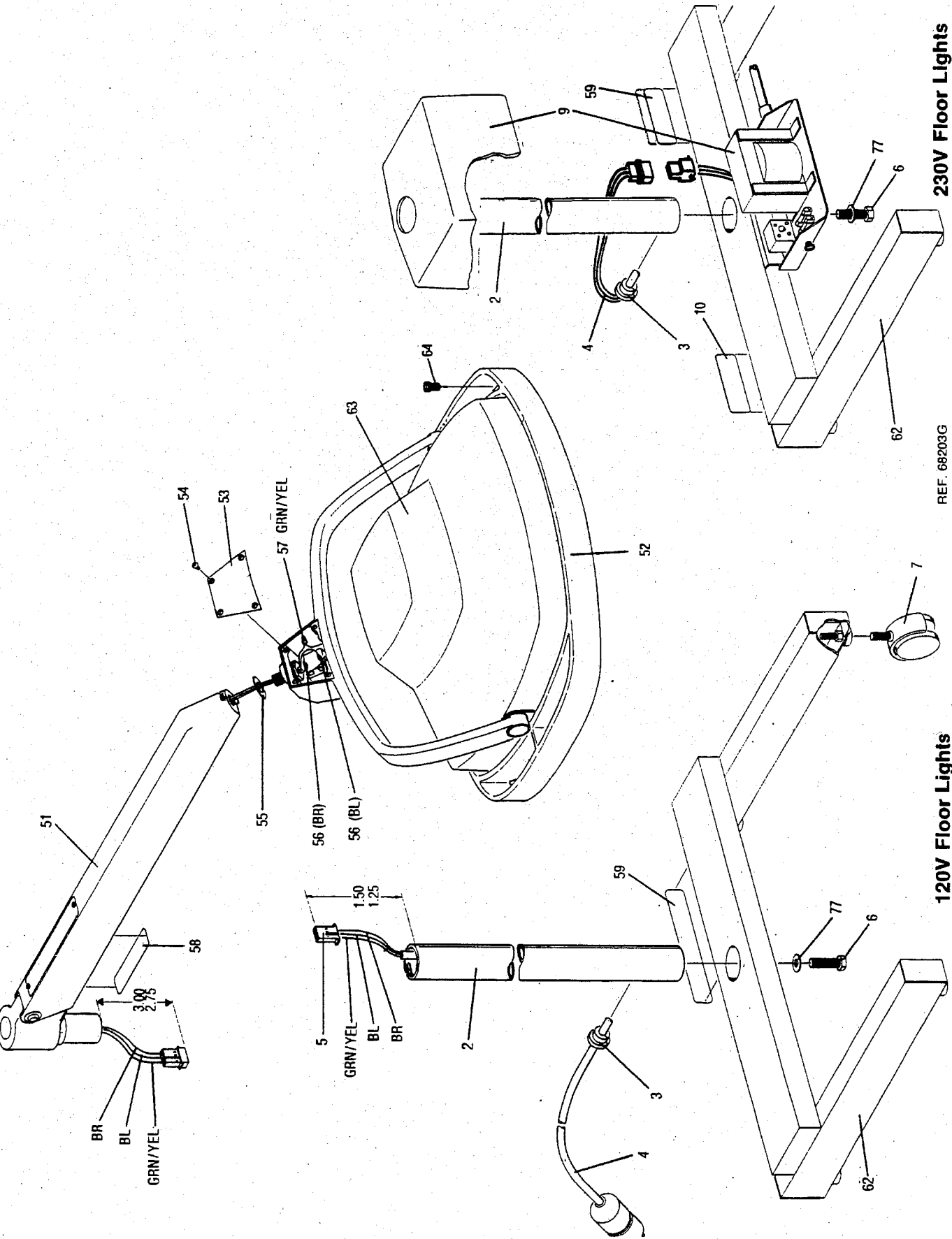

230V Floor Lights


REF. 68203G

120V Floor Lights

Floor Model Light Assembly — Refer to Figure 3

ITEM	PART NO.	DESCRIPTION
2	67647	Post Assembly, Floor Light Support
	67645	Post Assembly, Floor Light Support (Export)
3	57030	• Strain Relief, 21/64 - 23/64 dia Cable
4	65094	• Cord, Power Supply
	65708	• Cord, Power Supply (Export)
5	65047	• Plug, 3 Connector
6	E6918	• Screw, 3/8-16 x 3/4 Hex Head
10	5116100009	• Label, ETL-IEC
77	E2175	• Washer, 3/8 Flat
7	67629	Caster, Twin Wheel
9	—	Transformer Assembly (Export)
		• See Figure 10 for listing of parts
51	65110	Arm Assembly, Counterbalance
		• See Figure 8 for listing of parts
52	65085	Lighthouse Assembly, 100W (Examination)
	57651	Lighthouse Assembly, 150W (Surgical)
		• See Figure 6 for listing of parts
53	65175	Cover, Switch (Castle Surgical Light)
	511772	Cover, Switch (Castle Examination Light)
	—	Cover, Switch (Model 2300 Light)
54	E6876	Screw, #6-32 x 5/16 Self-Tapping Phillips Oval Head
55	65026	Washer, Keyed
56	65199	Terminal, Female Slip-On Flag
57	B0105	Nut, Wire (Two #18 Min - Two #16 Max)
	65724	Connector, Wire Butt
58	65141	Decal, Danger, Explosion Hazard
59	301110	Decal, Light Rating
62	67625	Base Assembly
	67990	Base Assembly (Export)
		• See Figure 9 for listing of parts
63	513915	Cover, Lighthouse
64	E6333	Screw, #6-32 x 3/8 Hex Socket Head
—	65196	Handle, Sterilizable (Optional)

SURGICAL & EXAMINATION LIGHTS
Figure 4. Ceiling Model Light Assembly

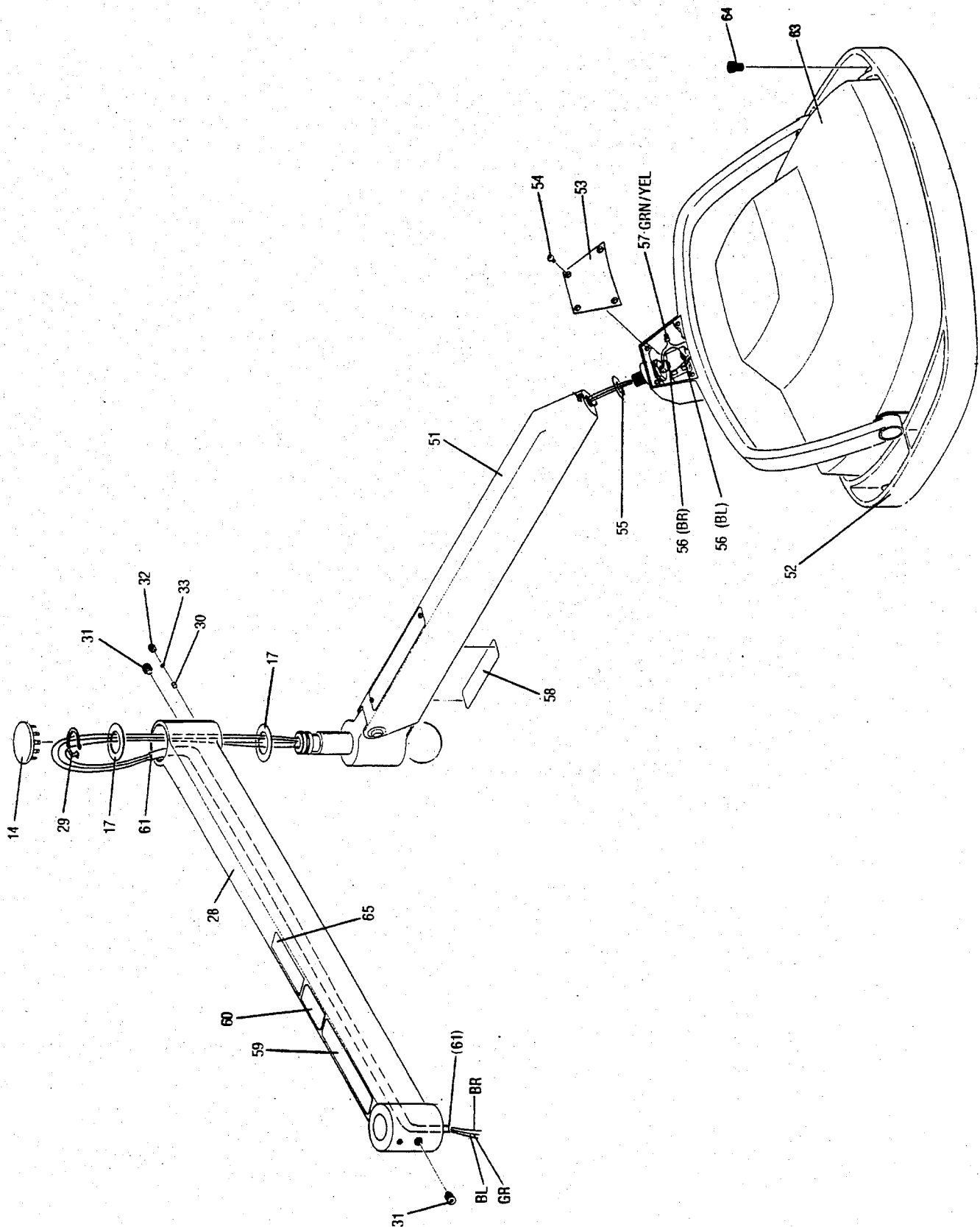


REF. 65117W

Ceiling Model Light Assembly — Refer to Figure 4


ITEM	PART NO.	DESCRIPTION
14	65059	Button, Plug for 1-7/16 dia Hole
17	65061	Race, Thrust, 7/8 ID
28	65160	Arm, Ceiling Light Extension (Brown)
	65161	Arm, Ceiling Light Extension (White)
29	E6207	Ring, External Retaining for 7/8 dia Shaft
30	43249	Plug, Friction, 3/16 dia x 1/8 long
31	E6244	Screw, 3/8-16 x 1/2 Half Dog Point Socket Set (with Nylon Insert)
32	E6981	Screw, 1/4-20 x 3/16 Flat Point Socket Set (with Nylon Insert)
33	65058	Washer, 15/16 Spring
51	65112	Arm Assembly, Counterbalance (Brown)
	57638	Arm Assembly, Counterbalance (White)
		• See Figure 8 for listing of parts
52	65085	Lighthouse Assembly, 100W (Examination)
	57651	Lighthouse Assembly, 150W (Surgical)
		• See Figure 6 for listing of parts
53	65175	Cover, Switch (Castle Surgical Light)
	511772	Cover, Switch (Castle Examination Light)
	—	Cover, Switch (Model 2300 Light)
54	E6876	Screw, #6-32 x 5/16 Self-Tapping Phillips Oval Head
55	65026	Washer, Keyed
56	65199	Terminal, Female Slip-On Flag
57	B0105	Nut, Wire (Two #18 Min - Two #16 Max)
58	65141	Decal, Danger, Explosion Hazard
59	301110	Decal, Light Rating
60	—	Label, UL Fixture
	5116100009	Label, ETL-IEC
61	R1922	Tubing, Plastic (specify length)
63	513915	Cover, Lighthouse
64	E6333	Screw, #6-32 x 3/8 Hex Socket Head
65	46018	Plate, Patent
—	65196	Handle, Sterilizable (Optional)

SURGICAL & EXAMINATION LIGHTS
Figure 5. Track Model Light Assembly

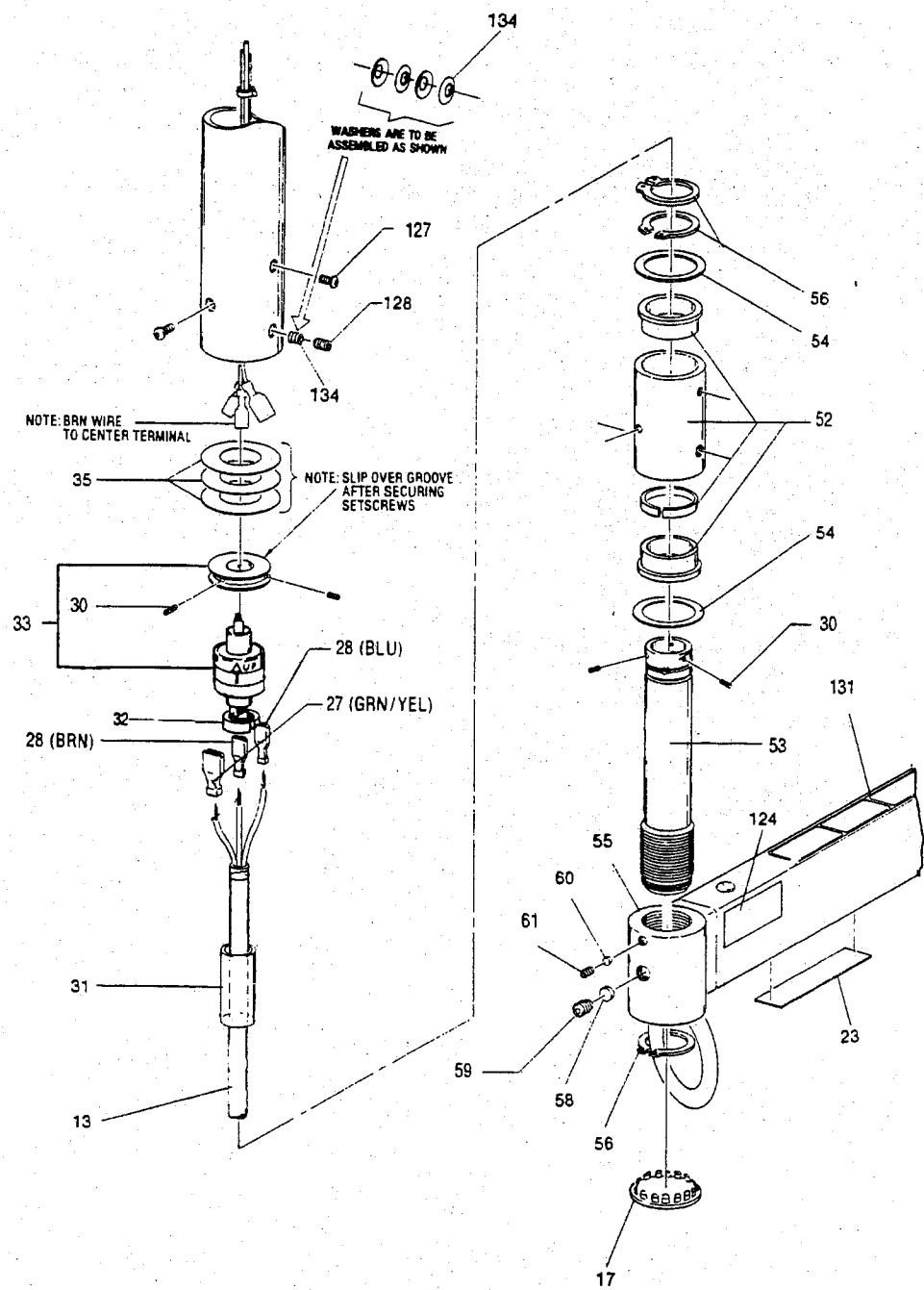

REF. 57649J

SURGICAL & EXAMINATION LIGHTS
Figure 5. Track Model Light Assembly

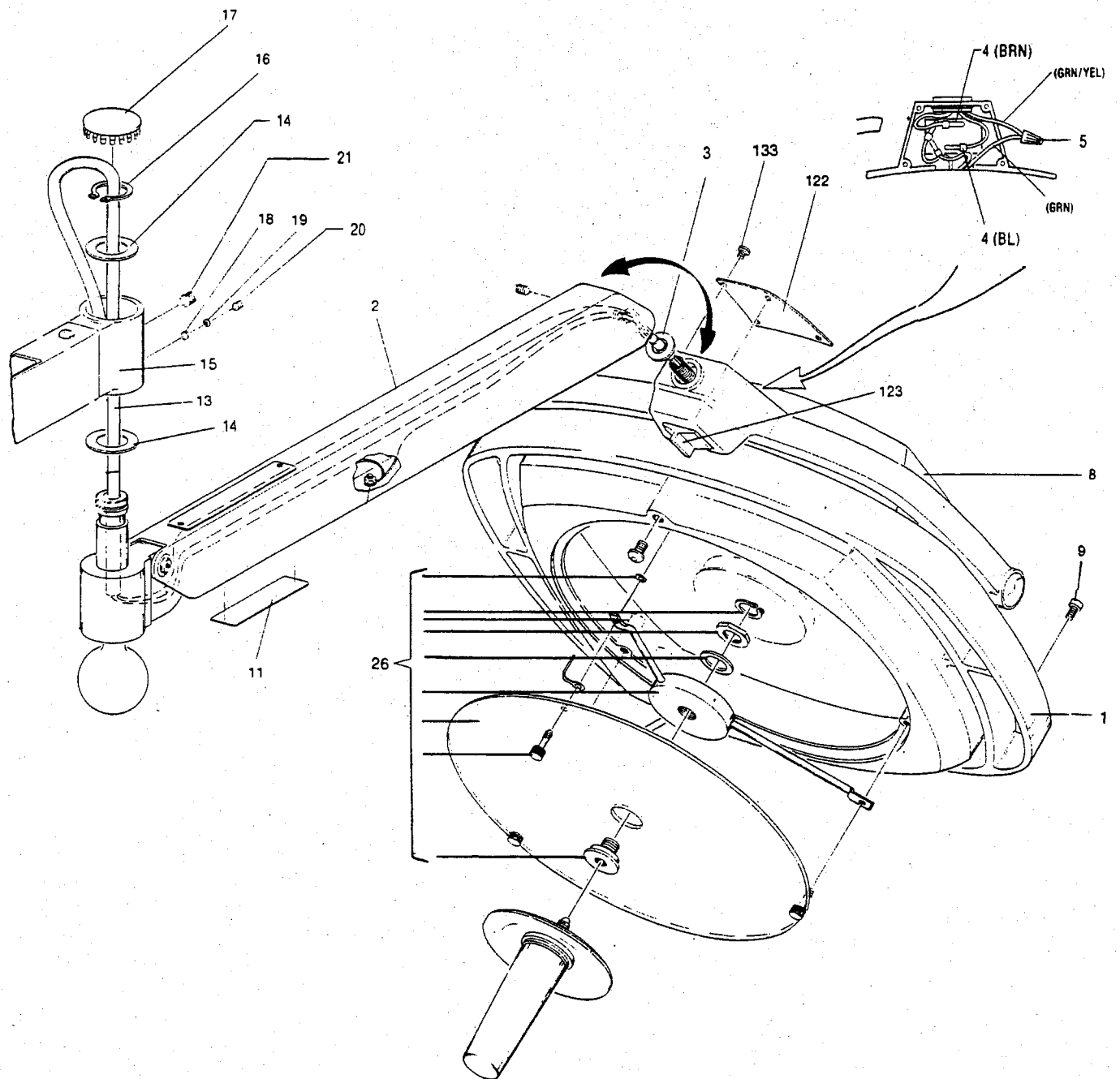

REF. 57649J

Track Model Light Assembly — Refer to Figure 5


ITEM	PART NO.	DESCRIPTION
1	57651	Lighthouse Assembly, 150W (Examination) <ul style="list-style-type: none">• See Figure 6 for listing of parts
2	57638	Arm Assembly, Counterbalance (White) <ul style="list-style-type: none">• See Figure 8 for listing of parts
3	65026	Washer, Keyed
4	65199	Terminal, Female Slip-On Flag
5	B0105	Nut, Wire (Two #18 Min - Two #16 Max)
8	513915	Cover, Lighthouse
9	E6333	Screw, #6-32 x 3/8 Hex Socket Head
11	65141	Decal, Danger, Explosion Hazard
13	R1922	Tubing, Plastic (specify length)
14	65061	Race, Thrust, 7/8 ID
15	57637	Arm Assembly, Extension (White) See Item Numbers 52 thru 65 for listing of parts
16	E6207	Ring, External Retaining for 7/8 dia Shaft
17	65059	Button, Plug for 1-7/16 dia Hole
18	43249	Plug, Friction, 3/16 dia x 1/8 long
19	65058	Washer, 15/16 Spring
20	E6981	Screw, 1/4-20 x 3/16 Flat Point Socket Set (with Nylon Insert)
21	E6885	Screw, 3/8-16 x 1/2 Half Dog Point Socket Set (with Nylon Insert)
23	46018	Plate, Patent
26	57639	Lens Package <ul style="list-style-type: none">• See Figure 6 for listing of parts
27	68504	Terminal, 1/4 Female Slip-On (#22 - #18 Wire)
28	68499	Terminal, 1/8 Female Slip-On (#22 - #18 Wire)
30	E6980	Screw, #4-40 x 3/16 Cup Point Socket Set
31	57609	Bushing, Insulator
32	57616	Sleeve, Rotating Connector
33	514041	Rotating Connector Package
35	57610	Washer, Anti-Twist

Track Model Light Assembly — Refer to Figure 5

ITEM	PART NO.	DESCRIPTION
52	57631	Mount Assembly, Pivot Bearing
—	57619	• Mount, Pivot Bearing
—	57608	• Ring, Pivot Friction
—	68909	• Bearing, Pivot Pin
—	E6881	• Screw, 1/4-20 x 3/16 Gib Point Socket Set
53	57618	Pin, Suspension Arm
54	65061	Race, Thrust, 7/8 ID
55	57632	Arm, Track Light Extension (White)
56	E6207	Ring External Retaining for 7/8 dia Shaft
58	513736	Plug, Locking, 3/8 dia x 1/8 long (Nylon)
59	E7230	Screw, 1/2-13 x 1/4 Socket Jam
60	57641	Plug, Locking, 3/16 dia x 1/8 long
61	E6981	Screw, 1/4-20 x 3/16 Flat Point Socket Set (with Nylon Insert)
122	65175	Cover, Switch (Castle Surgical Light)
123	65077	Switch, ON/OFF Rocker
	65137	Switch, ON/OFF Rocker (Export)
124	5116100009	Label, ETL-IEC
127	25191	Screw, #10-32 x 3/8 Hex Socket Button Head (SST)
128	E6979	Screw, 5/16-24 x 5/16 Oval Point Socket Set
131	301110	Decal, Light Rating
133	E6876	Screw, #6-32 x 5/16 Self-Tapping Phillips Oval Head
134	69871	Washer, 1/8 ID x 1/4 OD Spring

SURGICAL & EXAMINATION LIGHTS

Figure 6. Lighthouse Assembly

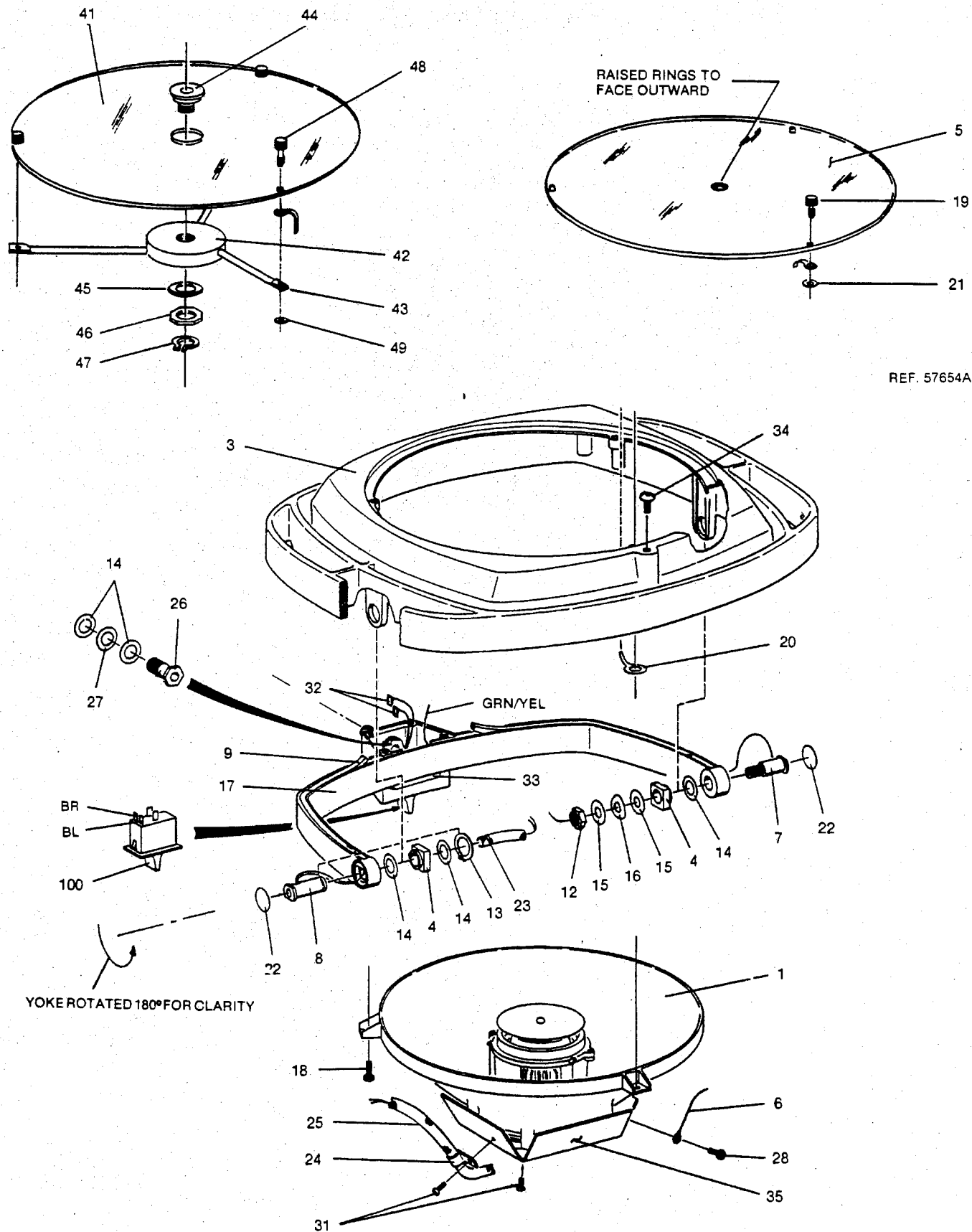

REF. 57654A

REF. 65084U

Lighthead Assembly — Refer to Figure 6 (less Rocker Switch)

65085 Lighthead Assembly - 100W (Examination)
57651 Lighthead Assembly - 150W (Surgical)


ITEM	PART NO.	DESCRIPTION
1	65082	Reflector Assembly, 100W (Examination)
	65083	Reflector Assembly, 150W (Surgical)
		• See Figure 7 for listing of parts
3	513916	Housing, Lighthead
4	65001	Bushing, Lighthead Pivot
5	65014	Lens, Diffusion
	67580	Lens Package (includes 1 Lens, 3 Thumbscrews & 3 Retaining Rings)
6	65088	Wire Assembly, Ground
7	65031	Pin, Lighthead Pivot (Ground Wire)
8	65000	Pin, Lighthead Pivot (Lamp Wires)
9	65063	Strip, Yoke Gasket
12	E6877	Nut, 3/8-24 Lock
13	E6894	Ring, Retaining for 1/2 dia Shaft
14	E5325	Washer, 1/2 Flat
15	E0245	Washer, 3/8 Flat
16	E0188	Washer, 3/8 Spring
17	65134	Yoke, Lighthead Mounting
18	E4821	Screw, #10 x 1/2 Self-Tapping Phillips Pan Head (Chrome)
19	65040	Screw, #8-32 x 3/8 Lens Retaining Thumbscrew
20	65039	Strap, Lens Retaining
21	E6290	Ring, External Retaining #8 dia Shaft
22	65045	Insert, Pivot Pin Trim
23	33511	Tie, Cable
24	43399	Clamp, 3/16 dia Cable
25	R3278	Tubing, Plastic (specify length)
26	65030	Pin, Yoke Pivot
27	E2002	Washer, 1/2 Spring
28	E6645	Screw, #10-32 x 3/8 Hex Head Ground
31	E5025	Screw, #8-32 x 5/16 Phillips Pan Head
32	65199	Terminal, Female Slip-On Flag
33	65198	Bumper, Stop
34	E6342	Screw, 5/16-18 x 1/2 Hex Socket Button Head (SST)
35	65019	Plate, Heat Dissipation

SURGICAL & EXAMINATION LIGHTS
Figure 6. Lighthouse Assembly

Lighthouse Assembly — Refer to Figure 6

ITEM	PART NO.	DESCRIPTION
41	57614	Lens, Diffusion
	57639	Lens Package (includes 1 Lens, 3 Thumbscrews, 3 Retaining Rings, 3 Rods & Mounting Hardware)
42	57613	Plate, Sterilizable Handle Mounting
43	57612	Rod, Sterilizable Handle Mounting
44	67552	Bushing, Handle Mounting Adapter
45	21526	Washer, 7/16 ID x 7/8 OD x 1/32 thick
46	E6921	Nut, 7/16 - 20 Hex
47	E5365	Ring, External Retaining for 3/8 dia Shaft
48	57611	Screw, #8-32 x 7/16 Lens Retaining Thumbscrew
49	E6290	Ring, External Retaining #8 dia Shaft
100	65077	Switch, ON/OFF Rocker
	65137	Switch, ON/OFF Rocker (Export)
-	65196	Handle, Sterilizable (Optional)

SURGICAL & EXAMINATION LIGHTS
Figure 7. Reflector Assembly

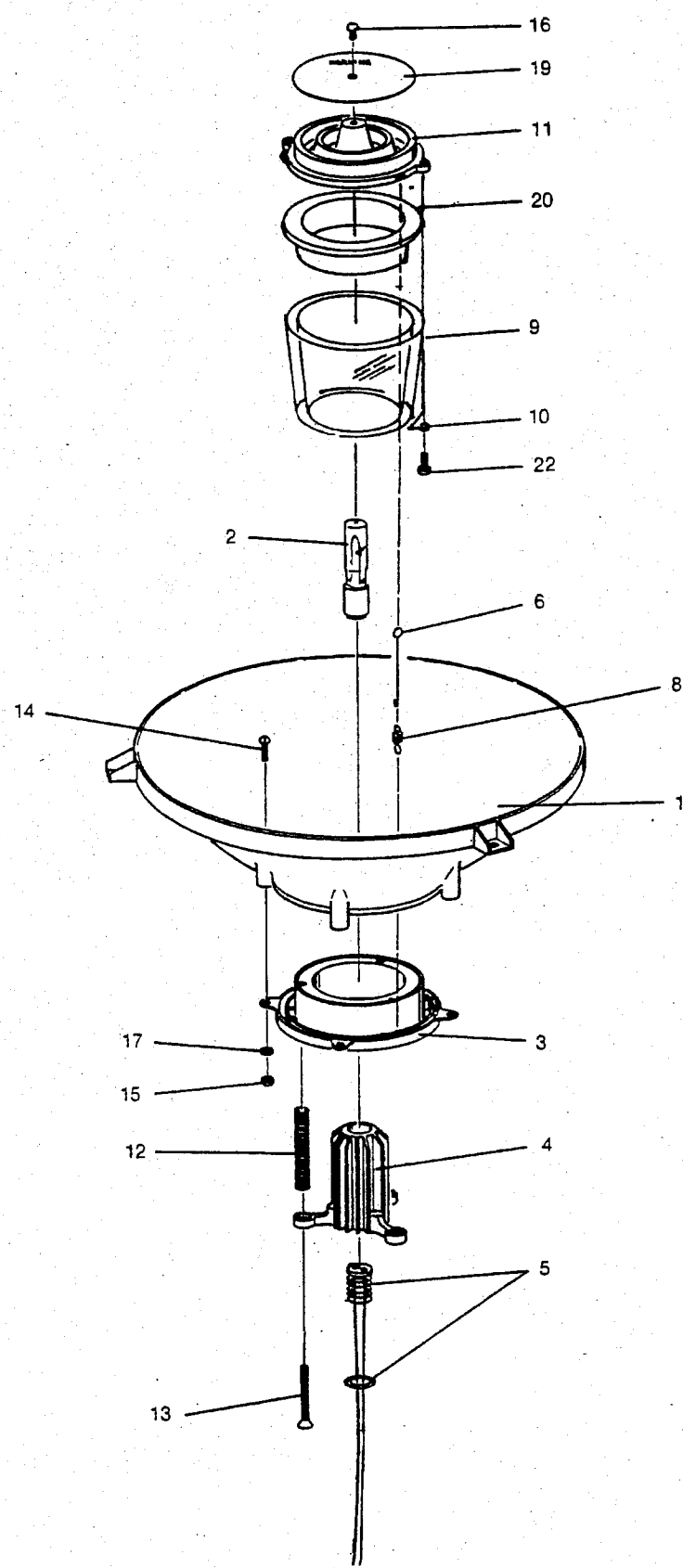


REF. 65084U

Reflector Assembly — Refer to Figure 7

65082 Reflector Assembly - 100W (Examination)
65083 Reflector Assembly - 150W (Surgical)

ITEM	PART NO.	DESCRIPTION
1	65011	Reflector, Aluminized (Examination Light)
	65012	Reflector, Dichroic (Surgical Light)
2	60976	Lamp, 120V-100W
	49455	Lamp, 120V-150W
3	65004	Heat Sink, Reflector Mounting
4	65003	Heat Sink, Lamp Socket
5	67501	Lamp Socket Package
6	65032	Hook, Baffle Retaining
8	65064	Spring, Extension, 1/4 dia x 5/8 long
9	61853	Cylinder, Glass Heat Filter
10	65065	Clip, Glass Heat Filter Retaining
11	65002	Baffle, Glass Heat Filter Bottom
12	65062	Spring, Compression, 23/64 dia x 2 long
13	E6567	Screw, #10-32 x 1-1/2 Phillips Flat Head
14	E5015	Screw, #6-32 x 1/2 Phillips Pan Head
15	E5180	Nut, #6-32 Hex
16	E2194	Screw, #6-32 x 1/4 Self-Tapping Phillips Pan Head (Chrome)
17	E7041	Washer, #6 Helical Spring Lock (SST)
19	65076	Disc, Baffle, 120V-100W Lamp
	65126	Disc, Baffle, 120V-150W Lamp
20	67553	Tube, Light Shield
22	E2169	Screw, #6-32 x 3/8 Self-Tapping Phillips Pan Head (Chrome)

SURGICAL & EXAMINATION LIGHTS **Figure 8. Counterbalance Arm Assembly**

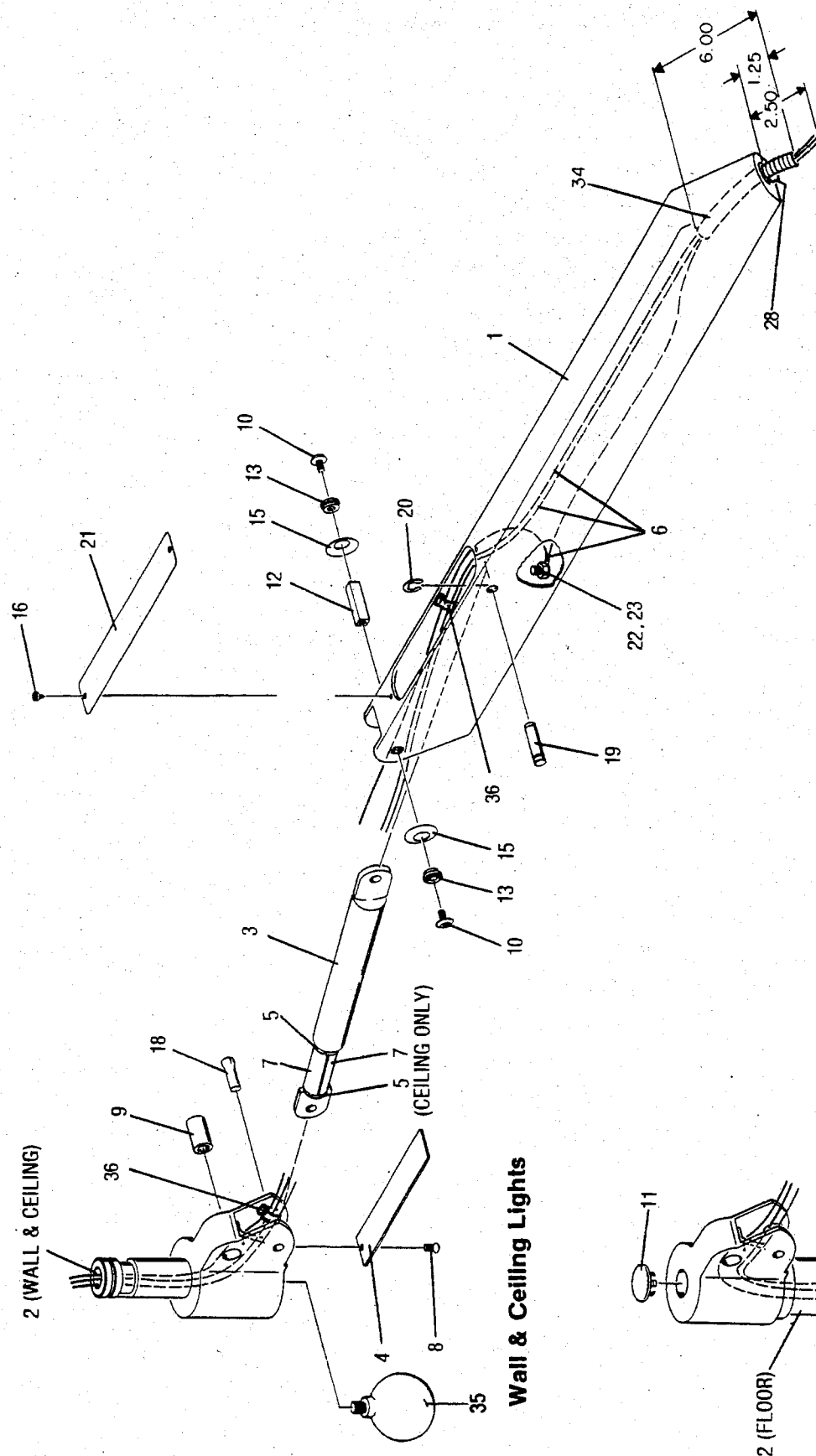

REF. 65139L

Wall & Ceiling Lights

Floor Lights


Counterbalance Arm Assembly — Refer to Figure 8

65111	Counterbalance Arm Assembly - Wall Light (Brown)
65172	Counterbalance Arm Assembly - Wall Light (White)
65110	Counterbalance Arm Assembly - Floor Light (Brown)
65112	Counterbalance Arm Assembly - Ceiling Light (Brown)
57638	Counterbalance Arm Assembly - Ceiling Light (White)

ITEM	PART NO.	DESCRIPTION
1	65169	Arm, Counterbalance (Brown)
	65170	Arm, Counterbalance (White)
2	57650	Mount, Counterbalance Arm (Brown)
	57634	Mount, Counterbalance Arm (White)
	65091	Mount, Counterbalance Arm (Floor Light)
3	65024	Spring, Gas
4	65028	Cover, Pivot Access (Brown)
	65142	Cover, Pivot Access (White)
5	65131	Washer, 3/8 Slotted
6	65108	Wire Assembly, Ground
	65107	Wire Assembly, Ground (Floor Light)
7	67480	Stop, Counterbalance Arm (Ceiling Light)
8	E6876	Screw, #6-32 x 5/16 S-Tapping Phillips Oval Head (Black)
	E6891	Screw, #6-32 x 5/16 S-Tapping Phillips Oval Head (Chrome)
9	65038	Bushing, Arm Pivot
10	25191	Screw, #10-32 x 3/8 Hex Socket Button Head
11	65060	Button, Plug for 11/16 dia Hole
12	65074	Pin, Arm Pivot
13	65075	Bushing, Trim
15	65080	Washer, 3/8 Spring
16	E6884	Screw, #4-40 x 3/16 S-Tapping Phillips Pan Head (Black)
	E0167	Screw, #4-40 x 3/16 S-Tapping Pan Head (Chrome)
18	E6875	Pin, Groove, 1/4 dia x 3/4 long
19	65163	Pin, Spring Mounting (Black)
	65164	Pin, Spring Mounting (Chrome)
20	E2042	Ring, Retaining for 1/4 dia Shaft
21	65145	Cover, Gas Spring Access (Brown)
	65146	Cover, Gas Spring Access (White)
22	E2044	Washer, #10 Internal Tooth Lock
23	E6886	Nut, #10-32 Hex Ground (Green)
24	R3264	Wire, #18 AWG Stranded, Blue (specify length)
25	R3263	Wire, #18 AWG Stranded, Brown (specify length)
26	65048	Connector, 3 Wire
27	65049	Contact, Pin
28	67484	Stop, Yoke Rotation
34	R1922	Tubing, Plastic (specify length)
35	57606	Knob, Ball, 5/16-18
36	45572	Tie, Cable


SURGICAL & EXAMINATION LIGHTS

Figure 9. Base Assembly (Floor Light)

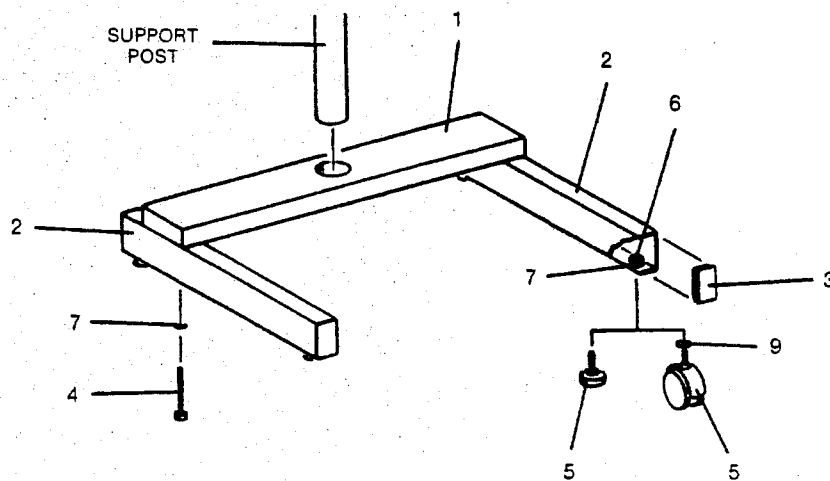

REF. 67625D

120V Floor Lights

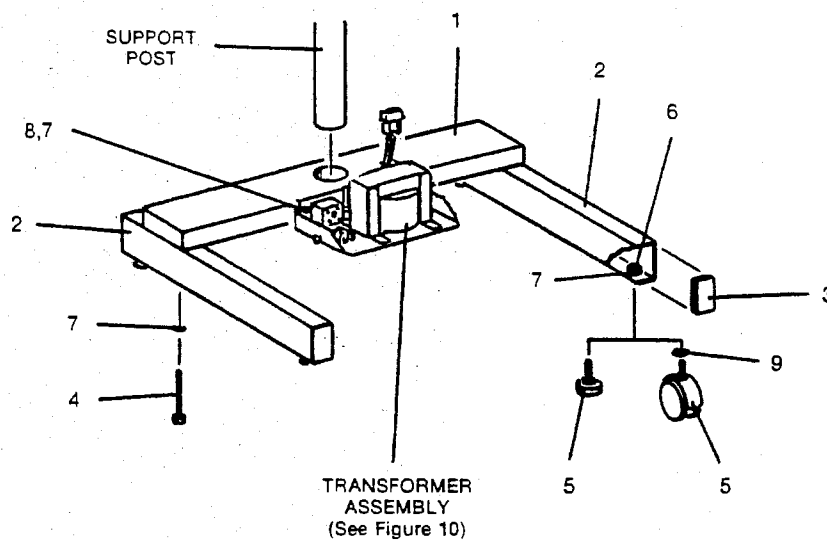

REF. 67990D

230V Floor Lights (Export)



Base Assembly (Floor Light) — Refer to Figure 9

67625 Base Assembly
67990 Base Assembly (Export)

ITEM	PART NO.	DESCRIPTION
1	67627	Plate, Top
	67649	Plate, Top (Export)
2	67626	Leg, Base
3	67628	Cap, Base Leg
4	E6919	Screw, 1/4-20 x 2-1/2 Hex Head
5	67630	Glide, Adjustable
	67629	Caster, Twin Wheel
6	E5186	Nut, 1/4-20 Hex
7	E5413	Washer, 1/4 Helical Spring Lock
8	E5536	Screw, 1/4-20 x 1 Hex Head
9	E2022	Washer, 1/4 Flat
—	—	Transformer Assembly (Export)
		• See Figure 10 for listing of parts

SURGICAL & EXAMINATION LIGHTS

Figure 10. Transformer Assembly (Floor Light - Export)

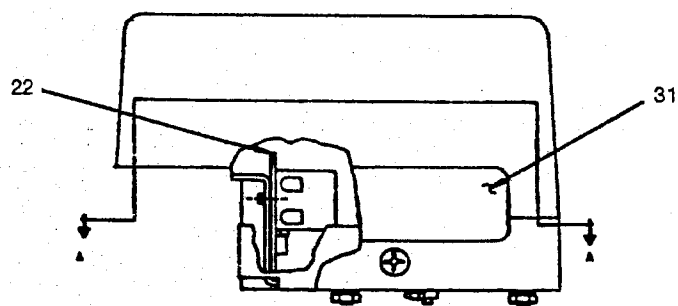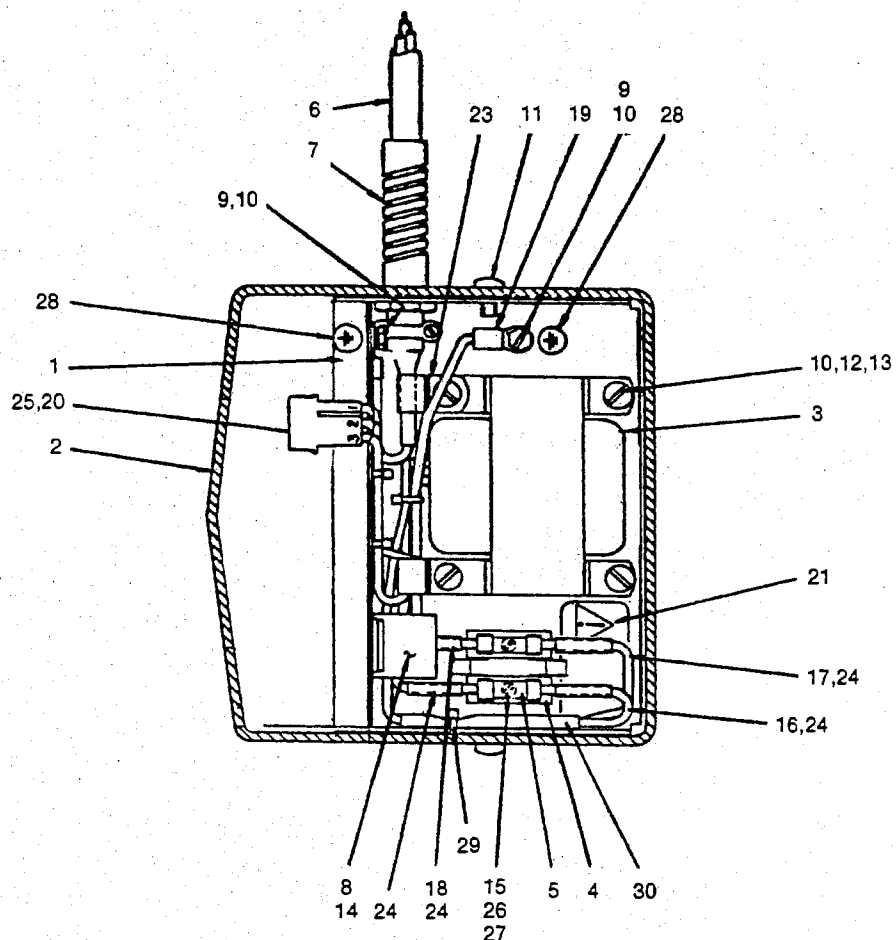


REF. 65709H

Transformer Assembly (Floor Light - Export)

Refer to Figure 10

	65709	Transformer Assembly - 100W (Examination)
	55919	Transformer Assembly - 150W (Surgical)
ITEM	PART NO.	DESCRIPTION
1	65705	Bracket, Transformer Mounting
2	65706	Cover, Transformer
3	65712	Transformer, 230V/120V, 50/60 Hz
4	65714	Holder, Fuse
5	65717	Fuse, 630mA, 250V (Examination)
	55918	Fuse, 800mA, 250V (Surgical)
6	65707	Cord, Power Supply
7	65716	Strain Relief, 25/64 dia Cable
8	65715	Block, Power Supply Terminal
9	E6645	Screw, #10-32 x 3/8 Hex Head Ground
10	E2044	Washer, #10 Internal Tooth Lock
11	E4840	Screw, #10-32 x 3/8 Self-Tapping Phillips Pan Head (Chrome)
12	E6924	Screw, #10-32 x 1/2 Phillips Pan Head
13	E0149	Nut, #10-32 Hex
14	E6540	Screw, #5-40 x 3/4 Round Head
15	E6610	Screw, #4-40 x 5/16 Phillips Pan Head
16	65710	Wire, Fuse (Brown)
17	65711	Wire, Fuse (Blue)
18	65720	Wire Assembly, Fuse
19	65713	Wire Assembly, Ground
20	65050	Contact, Socket
21	65718	Label, Fuse Warning (Examination)
	55920	Label, Fuse Warning (Surgical)
22	67578	Insulation, Bracket
23	43399	Clamp, 3/16 dia Cable
24	52291	Tubing, Heat Shrinkable
25	65047	Socket, 3 Wire
26	E7027	Washer, #4 Helical Spring Lock
27	E7028	Nut, #4-40 Hex
28	65723	Label, Ground Identification (Protective Earth)
29	45572	Tie, Cable
30	R2088	Tubing, Plastic (specify length)
31	67579	Label, Caution

SURGICAL & EXAMINATION LIGHTS
Figure 11. Single Point Mounting Suspension Package

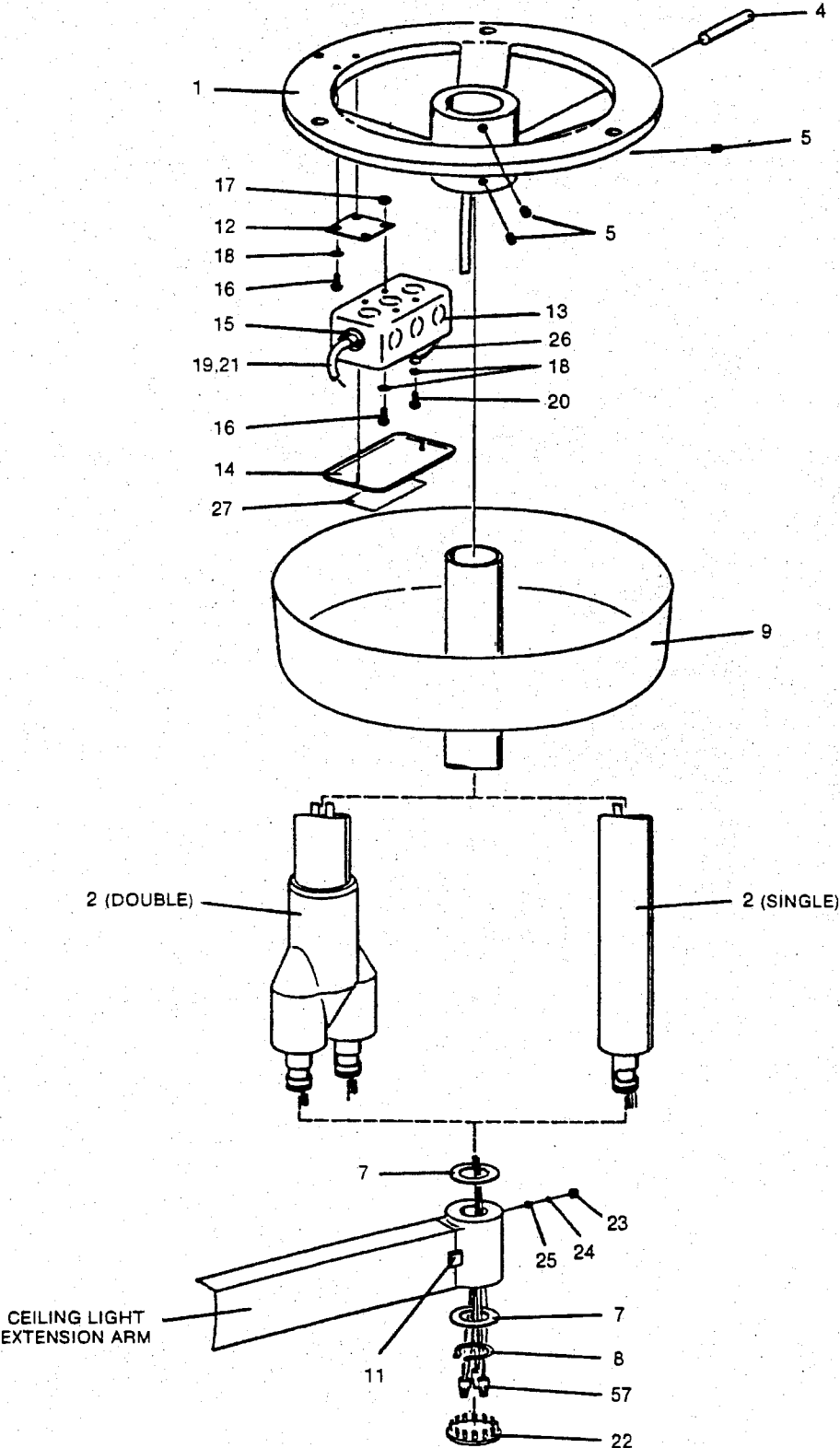


REF. 65130V

**Single Point Mounting Suspension Package — Refer to Figure 11
(less Suspension Tube & Canopy)**


	65122	Suspension Package (Single Light)
	65123	Suspension Package (Double Light)
ITEM	PART NO.	DESCRIPTION
1	65121	Tripod, Ceiling Suspension
2	65154	Tube, Single Light Suspension (Brown)
	65155	Tube, Single Light Suspension (White)
	68591	Tube, Double Light Suspension (Brown)
	65102	Tube, Double Light Suspension (White)
4	E4747	Pin, Groove, 3/8 dia x 2-1/2 long
5	E2142	Screw, 1/4-20 x 1/4 Cup Point Socket Set (Black)
7	65061	Race, Thrust, 7/8 ID
8	E6207	Ring, External Retaining for 7/8 dia Shaft
9	65017	Canopy, Tripod Cover
11	65144	Bumper, Protective 1/2 x 1/2 (White)
	68592	Bumper, Protective 1/2 x 1/2 (Brown)
12	65132	Bracket, Junction Box Mounting
13	38379	Box, Junction
14	03495	Cover, Junction Box
15	307026410	Strain Relief, 1/2 dia Cable
16	E4945	Screw, #10-24 x 3/8 Round Head
17	E0274	Nut, #10-24 Hex
18	E2044	Washer, #10 Internal Tooth Lock
19	R3262	Cord, 3 Wire #16 AWG Stranded (specify length)
20	E6645	Screw, #10-32 x 3/8 Hex Head Ground
21	87327	Nut, Wire (Three #20 Min - Three #16 Max)
22	65059	Button, Plug for 1-7/16 dia Hole
23	E6981	Screw, 1/4-20 x 3/16 Flat Point Socket Set (with Nylon Insert)
24	65058	Washer, 3/32 Spring
25	43249	Plug, Friction, 3/16 dia x 1/8 long
26	49106	Terminal, #10 Ring (#22 - #16 Wire)
27	65727	Decal, Electrical Connection
57	B0105	Nut, Wire (Two #18 Min - Two #16 Max)

SURGICAL & EXAMINATION LIGHTS
Figure 12. Trolley Assembly & Track Assembly

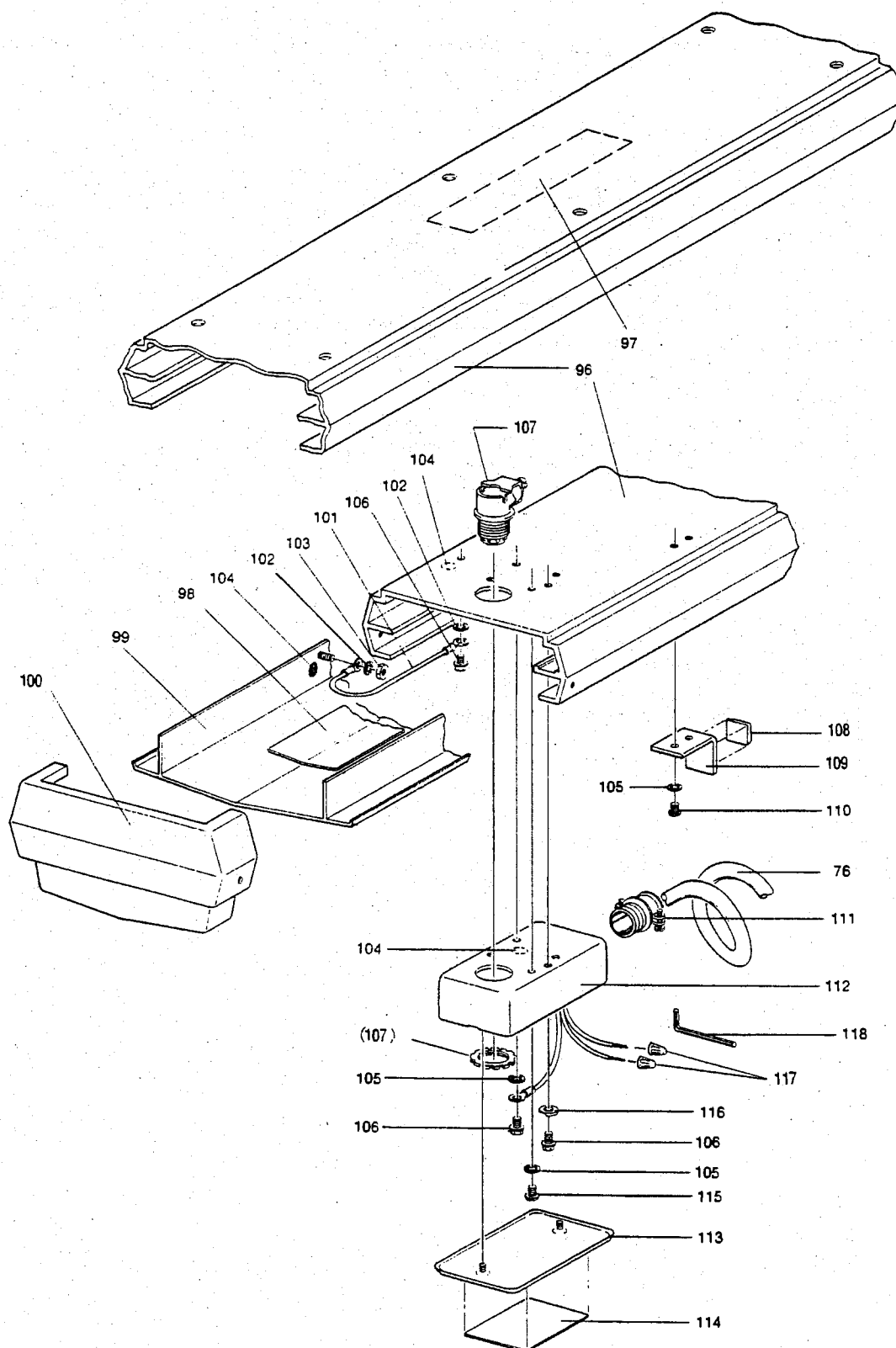

REF. 57649J

SURGICAL & EXAMINATION LIGHTS **Figure 12. Trolley Assembly & Track Assembly**

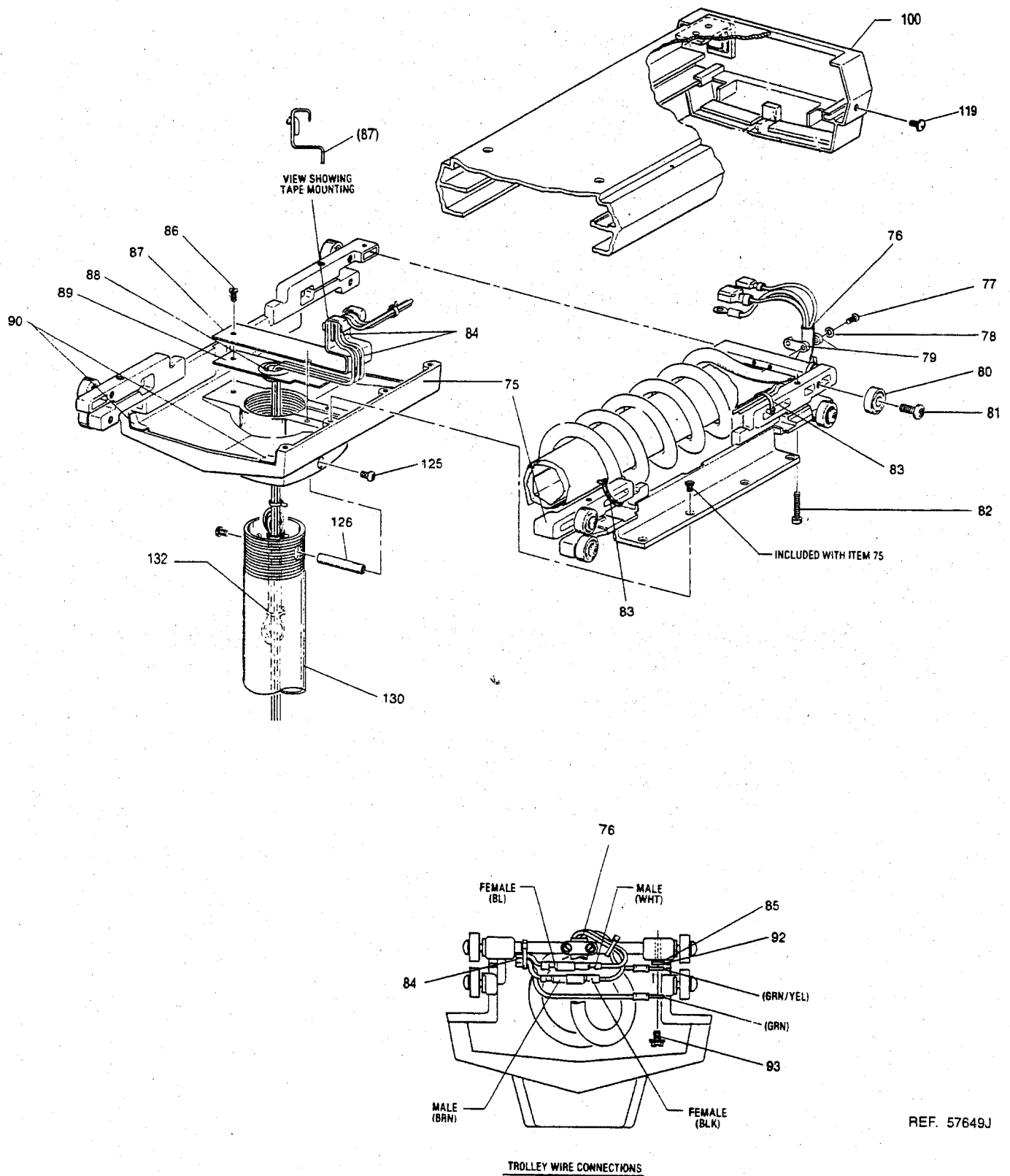

Trolley Assembly - 57647 — Refer to Figure 12


ITEM	PART NO.	DESCRIPTION
75	57584	Trolley Assembly
76	57587	Cord Assembly, Coiled
77	E6900	Screw, #4-40 x 3/4 Phillips Pan Head
78	E2053	Washer, #4 Internal Tooth Lock
79	67949	Clamp, Small
80	57601	Bearing, Ball, 1/4 ID x 3/4 OD
81	E4656	Screw, 1/4-20 x 3/4 Hex Socket Button Head (SST)
82	E6982	Screw, #10-32 x 1 Hex Socket (with Nylon Insert)
83	33511	Tie, Cable
84	57643	Harness, Trolley to Suspension Tube
85	65723	Label, Ground Identification (Protective Earth)
86	E6977	Screw, #6-32 x 1/4 Phillips Truss Head
87	57577	Cover, Wire
88	57594	Grommet, 13/32 ID x 13/16 OD for 1/2 dia Hole
89	57578	Bracket, Protective
90	57592	Pad, Felt, 15/16 x 3/4 x 3/64 thick
92	E2044	Washer, #10 Internal Tooth Lock
93	E6645	Screw, #10-32 x 3/8 Hex Head Ground

Track Mounting Package - 57659 — [Not Illustrated]

ITEM	PART NO.	DESCRIPTION
—	57658	Coupling, Track Leveling (3/8-16)
—	57646	Plate, Track Anchoring
—	E2135	Screw, 3/8-16 x 1 Hex Head
—	E5245	Washer, 3/8 Helical Spring Lock (SST)

Track Assembly - 57648 — Refer to Figure 12

ITEM	PART NO.	DESCRIPTION
96	57585	Track, Ceiling Model
97	57605	Label, Caution
98	R3473	Tape, 1/2 Wide Plastic
99	57581	Cover, Ceiling Model Track
100	57657	Cap, Track End
101	57604	Wire Assembly, Track Cover Ground
102	E6286	Washer, #10 Internal Tooth Lock
103	E5234	Nut, #10-32 Hex (Brass)
104	65723	Label, Ground Identification (Protective Earth)
105	E2044	Washer, #10 Internal Tooth Lock
106	E6645	Screw, #10-32 x 3/8 Hex Head Ground
107	07797	Connector, 3/8 dia Straight
108	57596	Bumper, Protective, 13/16 x 13/16 (White)
109	57582	Bracket, Trolley Stop
110	E6978	Screw, #10-32 x 5/16 Phillips Pan Head
111	13549	Strain Relief, 1/2 dia Cable
112	38379	Box, Junction
113	03495	Cover, Junction Box
114	57635	Decal, Electrical Connection
115	E4970	Screw, #10-32 x 1/4 Phillips Truss Head
116	61419	Washer, #10 Terminal Cup
117	B0105	Nut, Wire (Two #18 Min - Two #16 Max)
118	57595	Key, 5/32 Hex
119	E0073	Screw, #8-32 x 3/8 Phillips Truss Head (SST)
125	E6257	Screw, #8-32 x 3/8 Hex Socket Button Head
126	57593	Pin, Lock, 1/4 dia x 2 long
130	57621	Tube, 12" Suspension (8'-0" - 8'-3" Ceiling)
	57622	Tube, 15" Suspension (8'-3" - 8'-6" Ceiling)
	57623	Tube, 18" Suspension (8'-6" - 8'-9" Ceiling)
	57624	Tube, 21" Suspension (8'-9" - 9'-0" Ceiling)
	57625	Tube, 24" Suspension (9'-0" - 9'-3" Ceiling)
	57626	Tube, 27" Suspension (9'-3" - 9'-6" Ceiling)
	57627	Tube, 30" Suspension (9'-6" - 9'-9" Ceiling)
	57628	Tube, 33" Suspension (9'-9" - 10'-0" Ceiling)
132	33511	Tie, Cable

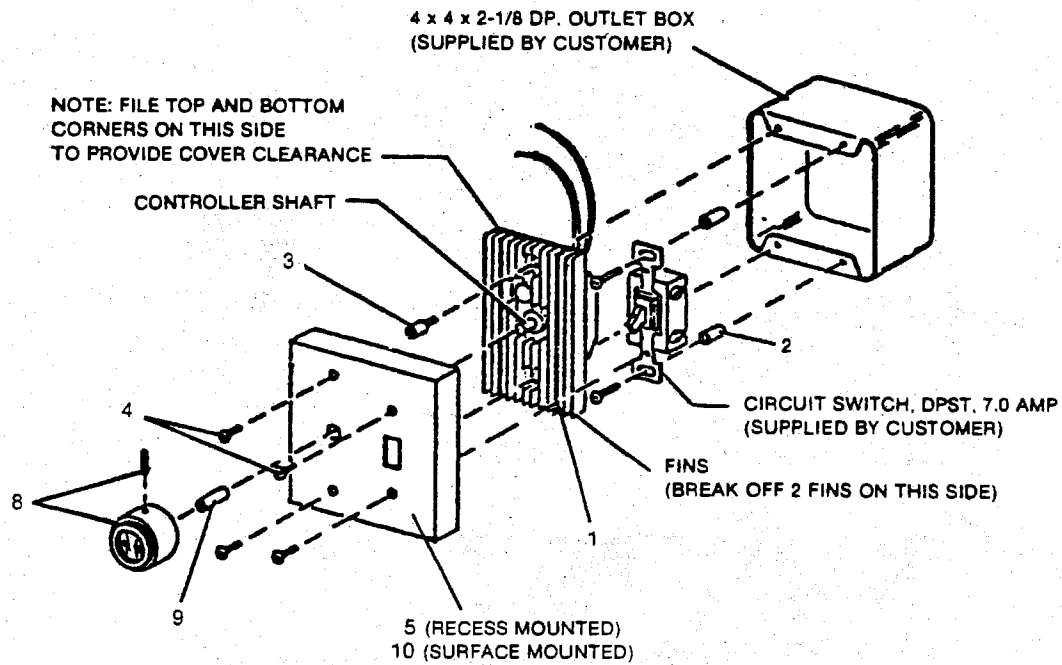


REF. 60881E

Dimmer Control Assembly (120V-600W) — 60881
Refer to Figure 13a

ITEM	PART NO.	DESCRIPTION
1	67969	Switch, Dimmer (120V-600W)
2	43756	Spacer, 3/16 ID x 1/4 OD x 1/2 long
3	43734	Spacer, Cover Mounting
4	E5010	Screw, #6-32 x 1/4 Phillips Pan Head
5	60892	Cover, Dimmer Control (Recess Mounted)
8	60909	Knob, 1-1/4 dia for 3/8 dia Shaft
9	60908	Adapter, Dimmer Switch Shaft
10	60893	Cover, Dimmer Control (Surface Mounted)
—	60667	Label, Use Copper Connectors

SURGICAL & EXAMINATION LIGHTS
Figure 13b. Light Control Assembly

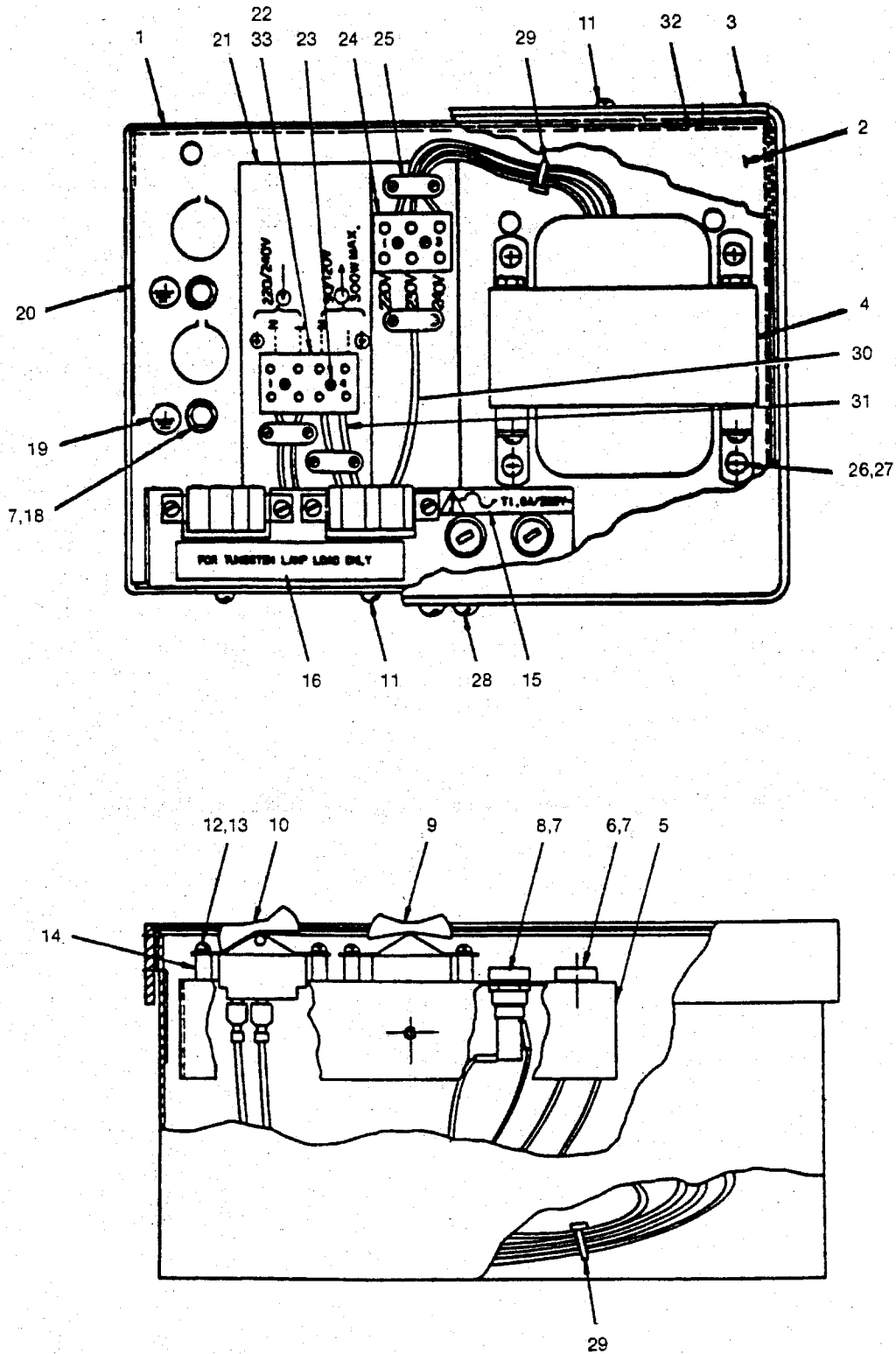

REF. 67194G

Surgical Light Control Assembly (230V-300W) — 67194
Refer to Figure 13b

ITEM	PART NO.	DESCRIPTION
1	67195	Box, Surgical Light Control (230V-300W)
2	67196	Cover, Control Box
3	42993	Strip, Trim
4	67941	Transformer, 220-240V/120-90V, 300W
5	67198	Bracket, Switch & Fuse Mounting
6	67602	Holder Assembly, Fuse (Brown)
7	67202	Fuse, 1.6A, 250V
8	67943	Holder Assembly, Fuse (Blue)
9	67200	Switch, 3 Position Rocker
10	67199	Switch, 2 Position Rocker
11	E6897	Screw, #6-32 x 1/2 Phillips Truss Head
12	E7185	Screw, #6-32 x 3/4 Phillips Pan Head
13	E4788	Washer, #6 Helical Spring Lock
14	67945	Spacer, 5/32 ID x 1/4 OD x 27/64 long
15	67946	Label, Fuse Rating
16	56530	Label, Tungsten Lamp Load
17	61419	Washer, #10 Terminal Cup
18	E6645	Screw, #10-32 x 3/8 Hex Head Ground
19	65723	Label, Ground Identification (Protective Earth)
20	301110	Decal, Light Rating
21	68348	Insulation, Terminal Block
22	67201	Strip, 4 Connector Terminal
23	E6900	Screw, #4-40 x 3/4 Phillips Pan Head
24	67948	Strip, 3 Connector Terminal
25	67949	Clamp, Small
26	E4941	Screw, #10-32 x 3/8 Phillips Pan Head
27	E2044	Washer, #10 Internal Tooth Lock
28	E6898	Screw, #10-32 x 1/2 Phillips Truss Head
29	33511	Tie, Cable
30	67206	Wire Assembly (Brown)
31	67205	Wire Assembly (Black)
32	5116100009	Label, ETL-IEC
33	60667	Label, Use Copper Connectors
-	E5210	Screw, #10-24 x 3/4 Self-Tapping Pan Head
-	38311	Plate, Control Box Trim
-	43003	Recess Rough-In Box Package

SURGICAL & EXAMINATION LIGHTS
Recommended Spare Parts List

Recommended Spare Parts List

It is recommended that the parts listed below be maintained in stock to ensure parts are immediately available when needed.

ITEM	PART NO.	DESCRIPTION
* 2	60976	Lamp, 120V-100W
* 2	49455	Lamp, 120V-150W
1	61853	Cylinder, Glass Heat Filter
1	65024	Spring, Gas
1	67969	Switch, Dimmer (120V-600W Control)
* 2	65717	Fuse, 630mA, 250V (Floor Examination Light - Export)
* 2	55918	Fuse, 800mA, 250V (Floor Surgical Light - Export)
2	67202	Fuse, 1.6A, 250V (230V-300W Control - Export)
2	65196	Handle, Sterilizable (Optional)
2	E6244	Screw, 1/2-13 x 1/2 Half Dog Point Socket Set (with Nylon Insert)
2	25191	Screw, Friction Adj., #10-32 x 3/8 Hex Socket Button Hd.

*Alternates – select part number that applies.

